

eUnity Measurement Tools



Linear Measurement – Click the left mouse button, drag the mouse and then click the left mouse button again to complete the length measurement.



Ellipse ROI (Region of Interest) – Select the tool then left click and drag the mouse to draw an ellipse.



Angle Measurement – Select the tool. Left click to create your start point. Move the mouse and then left click again to create a line. Move the mouse and left click again to complete the angle.



Freeform ROI (Region of Interest) – Select the tool and then left click as many times as required around the ROI. When finished, click the starting point to complete the freeform measurement.



Cobb Angle – Select the tool. A single left click starts the measurement and a second left click ends the measurement.



Eraser – Select the tool and then left click on measurement or markup to erase. Pressing the DELETE key on the host computer's keyboard will also erase the selected markup.



Revert to Original – Left click to reset a series of images to the original state.